


CHILI Diagnost

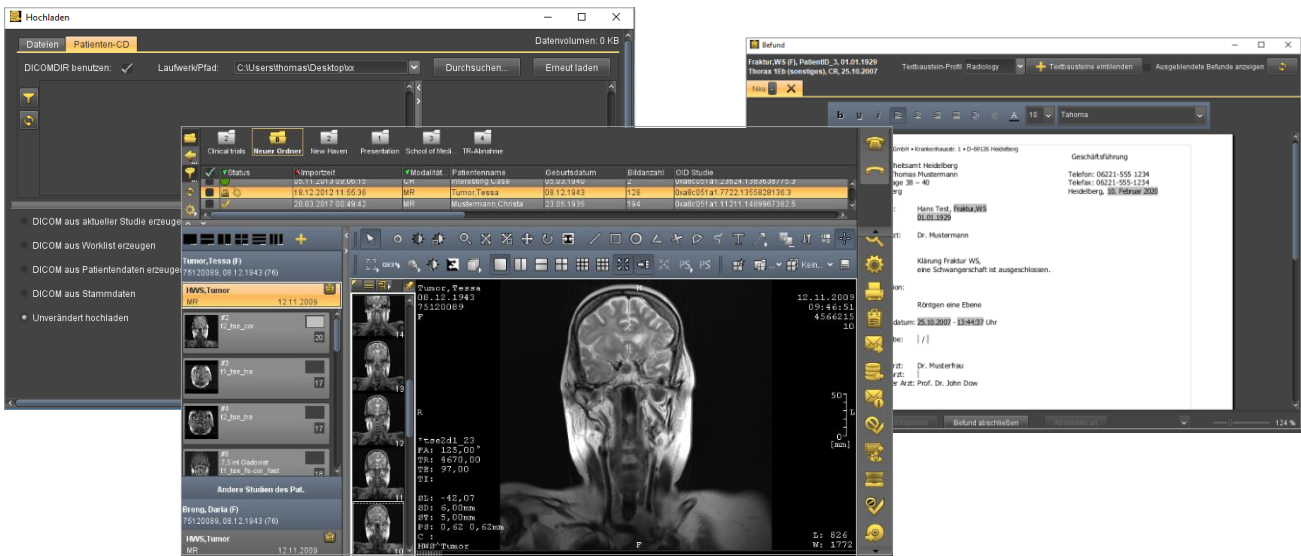

This quick reference does not replace the detailed user manual, which includes important safety information.




Selecting studies from database

Database functions

- Show folders
- Show study history
- Select and set filter
- Refresh database
- Arrange database

Database search

- Double-click in column header, enter a term, and press <Enter> to confirm

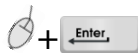

Show further column in database

- Right-click in header

Hide database and show full screen

- Press <Enter>



Arranging image areas

Series/previous studies



Pre-sets to arrange images in work space (1, 2, 3, or 4 image areas)



New image area

Changing the workspace

Teleconferenz

- Accept conference
- Call/hang up

Work tasks





- Images
- Data
- Setup
- Printer
- Report
- Send
- Import/Upload
- CD export
- Manual
- Splash-Screen

Quit



Displaying and editing images

Lightbox functions

-  Show only marked images
-  Localizer
-  Compression
-  Pin lightbox

Navigate in lightbox














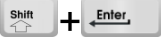













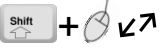
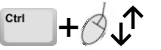
-   Browse up and down in column
-  Maximize active lightbox (full screen)
-   Browse image by image
-   Browse page by page
-  Mark image
-  Remove image marking
-  Display first image
-  Display last image

Image display

-  Marker
-  Sort by
-  Show/hide overlays
-  Select current zoom factor
-  Set level/window
-  Reset level/window
-  Invert image
-  Display mode
-  Number of displayed images (1, 2, 4, 6, or 9 images)
-  Fit to image window
-  Apply actions to all images
-  Reset all (rotate, zoom, pan, level/window)
-  Apply hanging automatically
-  Select profile
-  Select hanging
-  Save view

Short cuts to zoom and move

-  Move image with pressed left mouse key
-  Zoom image with pressed left mouse key or mouse wheel

Short cuts for image presentation


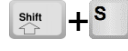
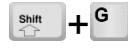


























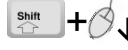

-  Edge enhancement on or off
-  Increase image sharpness on or off
-  Smoothing on or off
-  Adjust level/window automatically to optimal grayscale
-  Set level/window with right-click and mouse drag
-  Cancel image loading




Image editing

-  Pointer
-  View grayscale
-  Level/Window
-  Level/Window in rectangle
-  Magnifier
-  Zoom
-  Zoom rectangle
-  Move/Rotate
-  Shutter
-  Measure length
-  Rectangle
-  Circle
-  3-point-angle
-  4-point-angle
-  Polygon
-  Freehand line
-  Text annotation
-  Single-sided arrow
-  Link images using drag & drop
-  Cine-loop scroll
-  Show/hide localizer
-  3D cursor
-  Cancel drawing objects (ROIs)

With active zoom function

-  Scale size of the magnification window
-  Set zoom factor

Delete objects (ROIs)

-  Delete selected ROIs (created by user)
-  Delete all created ROIs
-  (Additional context)

NEXUS / CHILI GmbH

Friedrich-Ebert-Str. 2
69221 Dossenheim/Heidelberg, Germany

Phone +49 6221 18079-10
E-mail info@nexus-chili.com
Internet www.nexus-chili.com

Status: Februar 2020
CHILI PACS version 4.70
CHILI Diagnost version 4.46



The authors reserve the right to make changes at any time without prior notice. All rights reserved, in particular the rights to reproduction, distribution, and translation. No part of the software or the user manual may be reproduced or processed, copied or distributed by electronic means without the prior permission in writing from the authors.

Caution: US federal law restricts the herein described devices to sale by or on the order of a physician